Infusion Pump Analyzer

The Next Generation in Infusion Pump Analyzers is here

Features - IPA-3400 Series

- **♦ Smaller in Size Larger in Features**
- **♦** Faster in Operation
- **♦ Easier To Use**
- High Accuracy
- ♦ Large 7" Color Touch Screen
- 1,2,3 and 4 Channel Models Available (Field Upgradeable)
- User Swappable Fully Self Contained Flow Modules
- Calibration in Flow Modules
 No Need To be Down For Calibration or Service
- Smooth Dual Syringe System
 Eliminates Drain Cycle Inconsistencies
- Whisper Quiet Operation
- Auto Start
- Auto Test Sequences
- Built in Data Collection
- **♦ PDF Reports Available through BC Flow**
- Industrial Grade SS Pressure Sensor
- ♦ Performs All IEC 60601-2-24 Required Tests
- ♦ 10 uL to 1600 mL/Hr
- **♦ 4 USB Ports, 4 AUX Ports**
- Flash Drives, Barcode Scanners, Printers, Keyboard and Mouse Directly Supported
- **♦ PC Compatible**
- **♦** Configurable Pressure (mmHg, PSI, Bar, kPa)
- ♦ Large 32GB Internal Memory

Functions-

- ♦ PCA/Bolus
- Back Pressure Simulation
- **♦ Occlusion Alarm**
- **♦ Trumpet Curve Analysis (BC Flow)**
- **♦ Data Download to PC or Flash Drive**
- **♦** Customizable Test Templates (Built-In)
- ♦ Self-Cleaning Cycle



IPA-3400

Patent Pending

The IPA-3400 is the most compact, full featured four channel analyzer on the market.

It is a high accuracy, easy to use system that incorporates full touch screen control of all processes without the use of old fashioned buttons and knobs. This new cutting edge Patent Pending design uses a dual syringe stepper motor driven system that provides continuous monitoring of the fluid flow without the need to stop and perform intermittent drains like older technologies do. This provides a more realistic flow path for the Infusion Device under test and therefore more accurate readings. Also, independent stepper motor control of the custom designed ceramic valving allows the system to run not only more quietly and more smoothly, but it also allows for a bidirectional powered fluid flow for use in the built-in cleaning cycle.

The IPA-3400 has built in auto-sequence capabilities that allows the user to perform automatic test procedures. This allows specific test routines specified by various manufacturers to be run, which provides a significant time savings as well as reduces the risk of human error.





Easy access to modules for expansion and calibration.

SOFTWARE

BC Flow[™] software is provided with the IPA-3400 to allow PC Control, Display, Storage and Recall of system data.

A BCeTESTTM software module is available to allow full integration of the IPA-3400 into the BCeTESTTM system.



All test results are stored internally in the large 32 GB memory. They can also be downloaded to a USB flash drive or directly to a PC.

There are specific requirements in IEC 60601-2-24 for not only flow readings but back pressure simulation, bolus (PCA) measurements and occlusion alarm monitoring. All of these features are specifically built into the IPA-3400 with simple to use on screen selections.

The IPA-3400 is designed to hold up to 4 IPA-3900-FM flow modules. These modules are individually serialized and calibrated so that they may be moved from channel to channel and even unit to unit. Once installed they are recognized by the IPA-3400 and their Serial Number and NIST Traceable Calibration information are presented on the display and utilized in all data reporting.


The interchangeable modules provide the user with unprecedented flexibility in their IPA Testing System. There is no need to be down when the unit is due for calibration. Only the modules need to be calibrated. This also allows the rotating of modules or the use of a spare module, thus providing flexibility not available in other systems. The same is true for service. If there is a problem on one channel, only the module needs to be serviced, eliminating downtime.

The IPA-3400 display may be raised to gain full access to the four user swappable flow modules. No wiring or plumbing is required for module installation. Simply lift the display, remove the cover plate, loosen the retainer screw and slide the module out. All plumbing and electronics are self-contained and all electrical connections are made via a slide-in mating connector on the end of the module. Then just reverse the process for installation.

The IPA-3400 allows for easy field expansion. You can buy a 1 channel unit and later purchase additional modules that can be added in the field. Just plug them in and the system will automatically recognize the additional module(s), reconfigure itself and you are ready to go.

SCREEN VIEWS


Main Screen

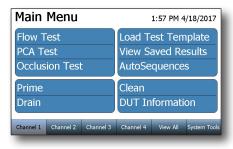


Flow Test **Settings**



Flow Test Data

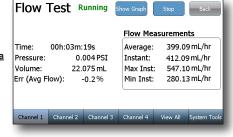

Flow Graph

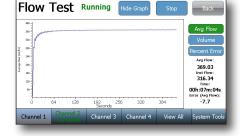

MODEL SUMMARY

BC Model	Description					
IPA-3400-1	Infusion Pump Analyzer Bench Top - Multi Channel w/ 1 Flow Module (IPA-3900-FM)					
IPA-3400-2	Infusion Pump Analyzer Bench Top - Multi Channel w/ 2 Flow Modules (IPA-3900-FM					
IPA-3400-3	Infusion Pump Analyzer Bench Top - Multi Channel w/ 3 Flow Modules (IPA-3900-FM					
IPA-3400-4	Infusion Pump Analyzer Bench Top - Multi Channel w/ 4 Flow Modules (IPA-3900-FM)					
Individual Components (Included)						
IPA-3900-FM	Flow module for the IPA-3400 series					
BC20-41360	Cable, Communication Null Modem for IPA-3400					
BC20-00161	IPA-3400 Accessory Kit					
tralia included	lorth America, Schuko, Ausd by default. Additional counterer cords are available:					
BC20-20501 Japan						
BC20-20502 United Kingdom						
BC20-20503 Schuko-Continental Europe						
BC20-20509 India/South Africa						
BC20-20510 Switzerland						
BC20-20512 Italy						

View All Channels

BC20-20516 Australia

Channel 1	FI	ow Test	Channel 2		Occ	lusion	Test	
Time:	00h:06m:48s		Current Pressure:		Current Pressure: -0.05		0.050	
Volume:	41.836	6 mL Current Time:			00h:00m:15s			
Average Flow:	368.99	1.476 Peak Pressure: 1.476 Peak Time: 00h:00m:06s						
Instant Flow:	311.92	mL/hr	Peak Time: Bolus Volum		00h:00			
Channel 3	Р	CA Test	Channel 4				Idle	
Time: 00h:03m:29s	Inst. Flow:	32.08						
Bolus Count:		2						
Bolus Flow (ml	/hr):	84.25						
Bolus Volume (mL):	0.773						
Bolus Duration		33						
Channel 1	annel 2	nannel 3	Channel 4	Viev	v All	System	Tools	

BC Group International, Inc. 308 Elm Point Industrial Dr, St. Charles, MO 63301 USA

Phone: (314)638-3800 Toll Free: 1(800)242-8428 Fax:(314)638-3200

Sales@BCGroupIntl.com www.BCGroupStore.com www.BCGroupIntl.com

IPA-3400 SPECIFICATIONS

Parameter		IPA-3400				
Display Range (ml/hr)		0.01 - 2600				
Flow Measurement	Flow Rate (ml/hr)	0.10 - 1600				
	Flow Resolution (ml/hr)		0.010 (10 μL)			
	Accuracy	1% rdg + 0.005 mL/hr				
	Min Volume (ml)	0.05 (50 µL)				
	Channels	1, 2, 3 or 4 (user-installable)				
Volume Measurement	Volume Range (mL)	0 to 9999				
	Volume Resolution (mL)	0.001 (1.0 μL)				
	Volume Accuracy	1% rdg after 100 μL				
	Display Range (mL)	0.1 to 100				
DCA/Dalva Magazinamant	Measurement Range (mL)	0.5 to 100				
PCA/Bolus Measurement	Accuracy	+/- 1%				
	Min Bolus Volume (mL)	0.01 (10 µL)				
	Range	0 - 120 Hours				
Elapsed Time	Resolution	1 Second				
	Accuracy		0.5 Second			
	Range	-2	258.57 to 2585.75 mmHg (-5 to 50 PSI)			
Occlusion (Pressure) Test	Resolution	0.05 mmHg (0.001 PSI)				
	Accuracy		0.1% FS			
	Range	-200 to 600 mmHg (-3.867 to 11.602 PSI)				
Back-Pressure Control	Resolution	0.05 mmHg (0.001 PSI)				
	Accuracy	0.1% FS				
Pressure Units	Selectable	mmHg, PSI, Bar, kPa				
	Fluid Fittings	Inlet Female Luer				
		Drain	in Male Luer Lock			
		Ports	4 x USB-A Host ports			
Interface	USB	Supported Devices	HID-compliant Keyboard, Mouse & Barcode Scanner Printers Flash Disks			
			Bluetooth module			
	AUX	Ports	4 (1 per channel)			
		Supported Devices	Nurse Call PCA trigger output (optional accessory cable)			
	Display	7" Color Touch Screen (800 x 480)				
Power Supply	Voltage Input - 90 to 264 VAC, 50/60 Hz, 90VA					
Weight	1 chan < 8 lbs 2 chan < 10 lbs 3 chan < 12 lbs 4 chan < 14 lbs					
Size (H x W x D)	7.8 (H) x 9.1 (W) x 10.2 (D)					
Storage Temperature	0 to 50 °C					
Operating Temperature	15 to 40 °C					
Data Storage	Internal 32 GB					